WORK1

Electric Penis Pump

Instructions

Introduction

The MACHO WORK 1 is an automatic penis pump with stepless suction adjustment. To use, place the cylinder over the penis and start on the lowest setting. The indicator light will turn blue at the lowest pressure and red at the highest. Press the "MACHO WORK" button to begin, and press it again to pause once you reach your desired pressure. To release built-up pressure, simply press the "RELEASE" button.

Rechargeable via the included USB cable, the MACHO WORK 1 is not waterproof in the electric section but can be fully disassembled for easy cleaning.

Packing list

- 1 x Penis Pump
- 2 x Seal Rings
- 1 x Charging Cable
- 1 x Instructions

Technical Parameter

- Material: Silicone, ABS, GPPS
- Vacuum Function: Electric
- Maximum Vacuum Pressure: -50kPa
- Working Voltage: 3.7V
- Battery Capacity: 420mAh
- Working Noise: <60dB
- Duration: >60 minutes (full power state)

Button Instruction

1. Long press for 2 seconds to power on/off.
2. Press once to activate or pause the vacuum function.
3. The indicator light changes color with the vacuum pressure intensity. (Blue indicates the starting level, red indicates the highest level.)

Hold this button to release the pressure.

Instructions for Use

1. Before using the product for the first time, please charge it. Please insert the provided USB charging cable correctly into the charging port of the product. Then you can charge the product using a computer USB interface or a mobile charger with a USB interface. During normal charging, the product indicator light flashes to indicate charging; when the battery is fully charged, the indicator light stays on continuously.
2. Apply water-based lubricant to the entrance of the penis or sealing ring.
3. Insert the flaccid penis into the vacuum pump through the sealing ring entrance.
4. Press and hold the power button for 2 seconds to turn on the device. Press once again to activate the vacuum function.
5. The vacuum will begin, which may be accompanied by a mild pulling sensation.

6. Once the penis is erect, immediately press the button to stop the vacuum, then press and hold the release button to discharge the pressure.
7. If you experience pain or discomfort, immediately press and hold the release button to discharge the pressure and stop using the device.

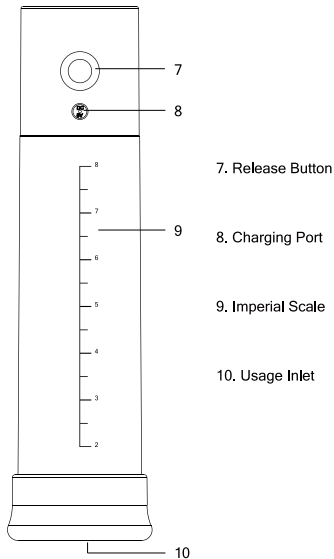
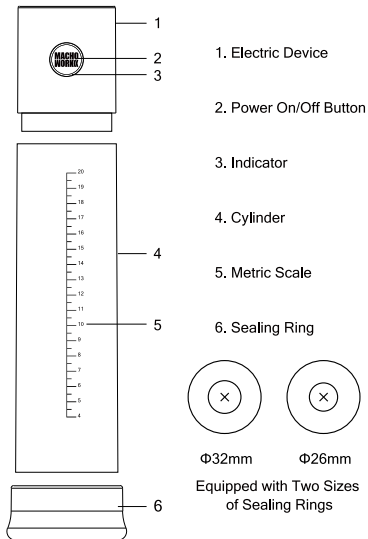
Cleaning and Maintenance

1. Before and after each use, clean the sealing ring with warm water and antibacterial soap.
2. Gently wipe off moisture with a dry towel and allow it to air dry naturally.
3. The device is not waterproof. Do not allow water to seep into the electric device.
4. Only use water-based lubricants. Do not use silicone-based lubricants.
5. Store in a clean, cool place, and keep out of reach of children.
6. When cleaning the device, unscrew the electric device and clean the cylinder and sealing ring.

Health and Safety Precautions

- The device cannot be operated while charging. Do not power on the device during charging.
- Only apply the minimum vacuum pressure necessary for achieving an erection. Excessive use may cause injury to the penis.
- If you experience pain, stop using the vacuum pump immediately.
- Do not use under the influence of alcohol or drugs, as they may impair your judgment and increase the risk of injury to the penis.
- If you have sickle cell anemia, a history of prolonged erections, or are taking large amounts of aspirin or other blood thinners, do not use the device to avoid increasing the risk of bruising and hematomas.
- The use of this device may cause abrasions or damage to the blood vessels in the penis or scrotum, leading to bruising, bleeding, or hematomas.
- The device may worsen existing conditions such as Peyronie's disease (formation of hardened tissue in the penis, typically causing pain, bending, and twisting during an erection), abnormal erections (persistent, often painful erections unrelated to sexual arousal), or urethral stricture (narrowing of the urethra, which can cause difficulty urinating).
- Incorrect use of the vacuum pump can cause damage to the penis.
- If you experience erectile dysfunction, such as the inability to achieve an erection sufficient for intercourse, consult your doctor before using this device to avoid harmful delays in diagnosing common underlying causes, such as diabetes, multiple sclerosis, cirrhosis, chronic kidney failure, or alcoholism.
- Do not use the product if you experience pain in the penis area.
- If you have reduced hand strength and cannot properly operate the product, do not use it.

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